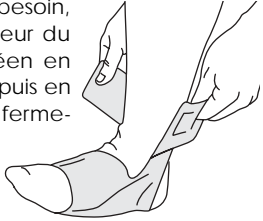
## Mode d'emploi :

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Sangle de cheville

### CARE

The entire pneumatic Achilles Wrap can be hand washed in luke warm water with mild soap and air dried.

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### HIGH ALTITUDE

At high altitudes the aircells will expand beyond their optimal level. Adjust aircell pressure by tightening or loosening strap.

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### LATEX

All Aircast products are latex-free.

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### WARRANTY POLICY

**Satisfaction** – Aircast will provide prompt refund for any product that does not satisfy the physician for any reason whatsoever.

**Durability** – The pneumatic Achilles Wrap is designed to last for as long as required by the original patient. Aircast may, at its discretion, furnish replacement parts during this time, provided the defective part is returned to Aircast for analysis.

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(09A-B)

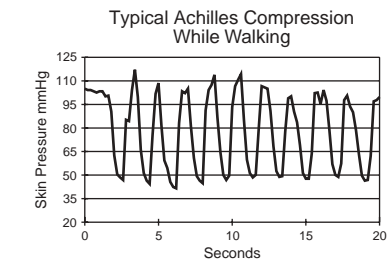
**AIRCAST**  
INCORPORATED

The **Aircast pneumatic Achilles Wrap** is intended for Achilles tendonitis. The wrap applies safe, intermittent compression to minimize swelling and promote circulation. This massaging compression allows for an early return to function with comfort and mobility. A low profile and lightweight design make it ideal for all levels of activity.

### OPERATION


With every step, air is compressed from the foot aircell to the Achilles aircell. Air flows back to the foot aircell between steps. This intermittent pulsating compression massages the tendon, enhancing circulation and reducing swelling.

## Instructions:

*The Achilles Wrap may be worn on either the left or the right foot.*

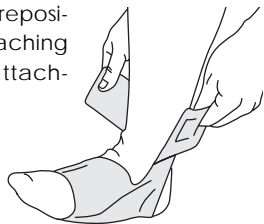- Put on sock. Insert foot, toe first into Achilles Wrap. Position heel in opening in back of wrap. Position Achilles aircell over tendon. The height of the Achilles aircell may be repositioned by detaching and firmly reattaching in place.

- Lift foot off floor and secure ankle strap. Make sure that foot aircell is located under arch of foot. To adjust Achilles Wrap fit, tighten/loosen ankle strap.

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