Instructions for use

Exoform[®] Ankle Brace

Life Without Limitations[®]

| | |
|-----------------------------------|----|
| | 3 |
| ⓔⓃ Instructions for use..... | 5 |
| ⓓⓔ Gebrauchsanweisung..... | 6 |
| ⓕⓗ Guide de fabrication..... | 7 |
| ⓔⓈ Instrucciones para el uso..... | 8 |
| ⓔⓣ Istruzioni per l'uso..... | 9 |
| Ⓝⓞ Bruksanvisning..... | 10 |
| ⓓⓐ Brugsanvisning..... | 11 |
| ⓈⓋ Bruksanvisning..... | 12 |
| ⓕⓙ Käyttöohjeet..... | 13 |
| Ⓝⓕ Gebruiksaanwijzing..... | 14 |
| ⓔⓣ Instruções de Utilização..... | 15 |

EN - Caution: This product has been designed and tested based on single patient usage and is not recommended for multiple patient use. If any problems occur with the use of this product, immediately contact your medical professional.

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ENGLISH

USES

Mild to moderate sprains and strains; injury prevention, chronically weak ankles.

CAUTION

1. Patients should be instructed to inspect their foot, ankle, and this product before and after each use.
2. This product is designed to be worn over an athletic sock. If any problems occur with this product please contact your certified medical professional. Measure ankle circumference over an athletic sock around the base of the heel and across the ankle bone to compare with the sizing chart to ensure proper product fit.

PHOTO CAPTIONS

- A. Speed Laces(1) with eyelets(2)
- B. Figure-8 Straps
- C. Tongue
- D. Heel opening

FITTING INSTRUCTIONS

1. Disconnect Figure-8 Straps(if applicable) and loosen laces. Insert foot into the brace toes first. Place heel securely into the heel opening. Adjust the tongue if necessary making sure the tongue is on top of the foot.
2. Lace up the brace using either the speed lace hooks(a) or the standard eyelets(b).
3. For version with Figure-8 straps: Wrap the inside (medial) strap over the top of the laces and underneath the foot.
4. Fasten the end of the strap on the loop material located on the inner side of the brace. Wrap the outside (lateral) strap over the top of the medial strap and underneath the foot. Fasten the end of the strap on the loop material located on the outer side of the brace.
5. Stand and apply full weight to the brace. Adjust straps to ensure a snug fit and maximum support.
6. Readjust laces as needed to ensure a supportive fit.

WASHING INSTRUCTIONS

1. This product may be cleaned using a mild detergent and cold water. Remove shoe laces and hand wash separately. Fasten straps as if in use. Do not use bleach or strong detergents.
2. Lay brace out flat on a towel and air dry. Do not squeeze or wring out the brace. Do not place in dryer or near heater. Inspect brace carefully after each washing.

Exoform Ankle ® is covered by one or more of the following patents. Other U.S. and foreign patents pending. ® indicates trademark registration in U.S.A. and selected countries only.

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